



APPOINTMENT #1: PRELIMINARY IMPRESSIONS & PROSTHESIS DUPLICATION

What you will need:

- Impression copings (available through Glidewell by request)
- Implant prosthetic driver
- X-ray bitewing tab
- Light-body, heavy-body and putty VPS
- VPS adhesive
- Acrylic bur
- Stock dentate impression tray
- Periphery wax
- Denture duplicating flask (available through Glidewell by request)
- Bite registration (rigid impression material)
- Camera

What you will send back to Glidewell:

- Preliminary implant-level impression
- Impression of opposing arch
- Bite registration with existing prosthesis in place
- Impression of existing denture or prosthesis
- List of any changes to existing denture or prosthesis
- Smile photo of the patient wearing existing denture or prosthesis
- Tooth shade selection

What you will receive next:

- Master cast with multi-unit abutments (if provided)
- Implant verification jig
- Custom impression tray
- Wax rim with six anterior teeth
- Prosthetic screws
- Opposing cast