

#### Step-by-Step Restorative Protocol

# THIRD APPOINTMENT

Try in the Wax Setup

glidewell.dental/bip-setup

NOTE: Before approving the wax setup and moving on to Fourth Appointment, make sure the esthetics and occlusion are 100% correct and verified. Two



RESET APPOINTMENT (If necessary)

Try in the Reset Wax Setup (if necessary)

Try in reset wax setup according to Third Appointment instructions.

Return the wax setup and list any adjustments on the Rx.

provisionals implant prostheses are included with the complete service; any additional provisionals will incur a fee of \$330 each.



If the patient is wearing a denture, remove the multi-unit abutment temporary healing caps. If the patient is wearing a fixed provisional, remove the prosthesis and reinsert the multiunit abutments.



Attach the wax setup directly to the multi-unit abutments with the enclosed prosthetic screws.



Confirm that the wax setup seats passively. Take a radiograph to confirm that the cylinders are fully seated.

### THIRD APPOINTMENT

Review checklist, making sure to complete each item. Make adjustments to the wax rim as necessary.

Midline
Incisal Edge Position
Lip Support
Smile Line/Occlusal Plane
Vertical Dimension of Occlusion
Phonetics
Clinical Smile Photo
Bite Registration (If Needed)



Verify the midline.



Verify incisal edge position and lip support.

#### THIRD APPOINTMENT



Verify the occlusal plane.



Verify the vertical dimension of occlusion (VDO).



Verify the patient's bite and take a new bite registration if necessary.

**NOTE:** If the bite is incorrect, grind or remove setup teeth to achieve an accurate bite.



Take clinical photos of the smile.

## THIRD APPOINTMENT

#### Try in the Wax Setup

NOTE: Before approving the wax setup and moving on to the Fourth Appointment, make sure the esthetics and occlusion are 100% correct and verified. Two provisionals implant prostheses are included with the complete service; any additional provisionals will incur a fee of \$330 each.



Remove the wax rim and bite registration. If the patient is wearing a denture, leave the multi-unit abutments in place, cover them with the enclosed temporary healing caps and relieve the intaglio surface of the denture to seat over the multi-unit abutments.

If the patient is wearing a fixed provisional, remove the multi-unit abutments and return them to their original positions on the master cast.



Select the gingival shade.

To purchase a BruxZir® Gingival Shade Guide, call Glidewell Direct at 888-303-3975 or visit glidewelldirect.com



Return the wax setup and list any adjustments on the Rx.

Have questions? Call 866-861-7955 or check out our how-to videos below.



BruxZir Implant Setup

glidewell.dental/bip-setup



For more information glidewell.com/bip 866-861-7955