

TAP 3[®] Elite Assembly Instructions



1. The TAP 3 Elite now includes a second bar on the mandible, making assembly much easier.



2. Line up the hook on the maxillary half and the space between the two bars on the mandibular half, with the labial facing the buccal as shown.



3. Lower the hook into the space.



4. Twist so that the labial portions line up.



5. Assembly of the TAP 3 Elite is complete.



**GLIDEWELL
LABORATORIES**

*If you have questions,
please call our technical staff.*

800-407-3326

GL-3536-0813