



# DIGITAL TREATMENT PLANNING (DTP)

## Quick Reference

- **How much are surgical guides?** Surgical guides are part of our Digital Treatment Planning services. Prices start at \$295 for the first implant site. The cost for each additional implant site within the same arch is \$100. We also offer a Hahn™ Guided Surgery System bundle for \$395 for the first implant site. This bundle includes a surgical guide, Hahn implant, healing abutment and impression coping or scan body. The cost for each additional implant site within the same arch is \$200.
- **How do I begin to set up a case with you?** Please digitally submit your case or download the Rx available at [glidewelldental.com/DigitalTreatmentPlanning](http://glidewelldental.com/DigitalTreatmentPlanning), or email your first and last name, office phone number (and account number if you know it), the patient's name, and the tooth number(s) corresponding to the site(s) needing guided implant placement to [DTP@glidewelldental.com](mailto:DTP@glidewelldental.com). Once we receive your email or prescription, a member of our Digital Treatment Planning department will contact you. To speak with a customer service representative, call 800-839-9755.
- **What do you need from me to make a surgical guide? How do I submit a case?** Obtain a CT/CBCT scan\* *and* full-arch digital impression or VPS impression. Submit your digital impression to us via your intraoral scanner software, noting in the comment section that the scan should be forwarded to the DTP department, or send your VPS impression via standard case shipping methods. CBCT data can be uploaded with a completed prescription at [glidewelldental.com/DigitalTreatmentPlanning](http://glidewelldental.com/DigitalTreatmentPlanning), or sent on disc via standard case shipping methods with a completed prescription. Note that your case can also be initiated by uploading your digital impression and CBCT scan through the My Account portal on [glidewelldental.com](http://glidewelldental.com).
  - \*Patient must be scanned with an open bite of at least 8-10 mm. This can be achieved by having the patient bite on a small stack of tongue depressors.
- **What file type does the CT data need to be saved in?** Your data needs to be in DICOM (DCM) file format. You will need to place the files into a folder, and then compress that folder so it can be uploaded. *Note: We cannot accept files that are in software format or (.DIR) file format.*
- **How do I review and approve the digital treatment plan?** A member of our team will send you a PDF to review and approve via email.
- **What is the turnaround time?** Once all necessary items are received from the clinician and the accuracy is confirmed, the digital treatment plan is developed and sent to the clinician for approval within 8 working days. Once the surgical plan has been approved by the clinician, the case ships within 5 days.
- **What implant systems are your surgical guides compatible with?**

Hahn™ Tapered Implant System  
BIOMET 3i™ Certain®  
CAMLOG® SCREW-LINE



**GLIDEWELL DENTAL**

DENTSPLY Implants Astra Tech OsseoSpeed® TX  
Nobel Biocare NobelReplace®  
Nobel Biocare NobelActive®  
Nobel Biocare Brånemark System®  
Straumann® Bone Level  
Straumann® Tissue Level  
Zimmer Dental Screw-Vent®

- **Which guided drill kits are compatible with your surgical guides?**

Hahn™ Tapered Implant Guided Surgery System  
BIOMET 3i™ Navigator®  
CAMLOG® Guide System  
DENTSPLY Implants Astra Tech OsseoSpeed® TX Guided Surgery System  
Nobel Biocare NobelGuide®  
Straumann® Guided Surgery System  
Zimmer Dental Tapered Screw-Vent® Guided Surgery System

- **How much is the Hahn™ Tapered Implant Guided Surgical Kit?** The kit costs \$3,295.
- **How do I purchase the Hahn Tapered Implant Guided Surgical Kit?** You can purchase the Hahn Tapered Implant Guided Surgical Kit by visiting [glidewelldirect.com](http://glidewelldirect.com) or calling Glidewell Direct at 888-303-3975.
- **Can I use my regular (non-guided) drill kit with a surgical guide?** Surgical guides require the use of a separate guided drill kit. Please check with your implant manufacturer to be sure you have the correct instrumentation to perform guided surgery.
- **Do you offer clinical advice on implant placement?** We offer surgical guides and digital treatment plans. The clinician should have the proper education and training prior to placing any implants.
- **Can I have a webinar with a technician?** Communication between our Digital Treatment Planning staff members and the clinician is done via email.
- **Can I have a technician assigned to me?** Our digital treatment planning staff members work very closely with each other. As a team, they will help you accomplish your goals for each case.
- **Do you offer bone-supported guides?** Our guides are tooth-supported. We do not offer tissue- or bone-supported guides at this time.
- **Do you offer pilot drill guides?** We offer pilot drill guides, or prosthetic stents, through the Implant department at Glidewell Laboratories. Prosthetic stents are model-based and give you a starting point for the pilot hole. They do not have metal sleeves. To obtain a prosthetic stent, you must send a study model to the lab.
- **Can I get an immediate provisional prosthesis for a guided full-arch case?** An immediate provisional can be made by Glidewell Laboratories (separate from the Digital Treatment Planning department), and would require a physical impression be sent into the lab before implant surgery.

*Hahn™ Tapered Implant is a trademark of PrismaDent Dentalcraft, Inc. All third-party trademarks are property of their respective owners.*