



BruxZir® Solid Zirconia Preparation Guidelines

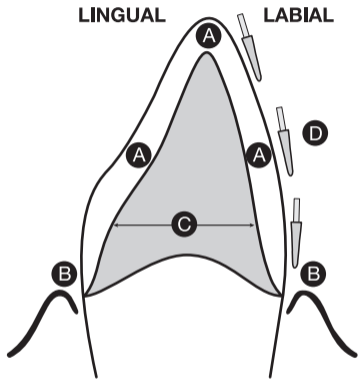

- A. 1.0 mm ideal reduction (0.6 mm minimum)
- B. Chamfer or shoulder margins preferred. Feather-edge OK
- C. Labial and lingual walls must be convergent
- D. Preparation should be cut in three planes

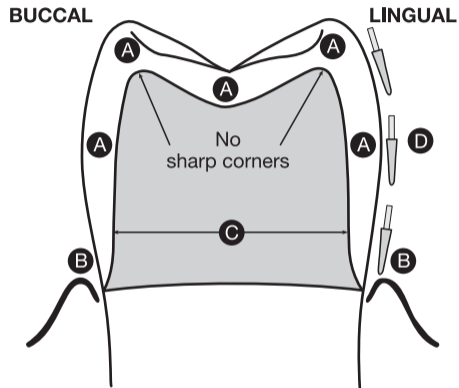

- A. 1.0 mm ideal reduction (0.6 mm minimum)
- B. Chamfer or shoulder margins preferred. Feather-edge OK
- C. Buccal and lingual walls must be convergent
- D. Preparation should be cut in three planes