

Full-Arch Mill Service
Step-by-Step Restorative Protocol





BruxZir
esthetic
FULL-ARCH IMPLANT PROSTHESIS



Glidewell's Full-Arch Mill Service (mill from final design file) produces a BruxZir® Full-Arch Implant Prosthesis from the digital design file for a full-arch screw-retained implant restoration. **For this service, the design file for the prosthesis must be fully approved. Design changes cannot be made as part of this service.**

■ FIRST APPOINTMENT

Upload Final Design File

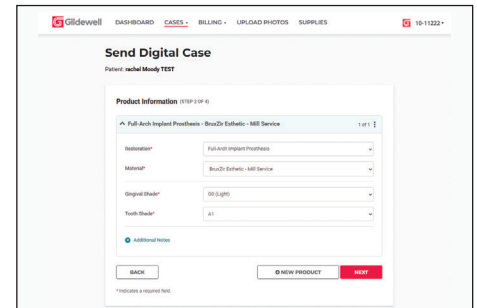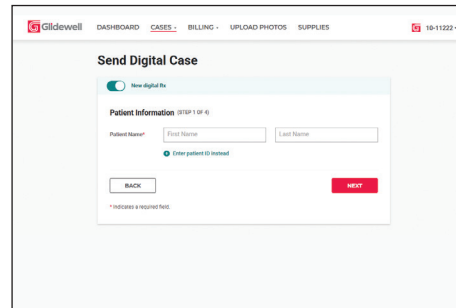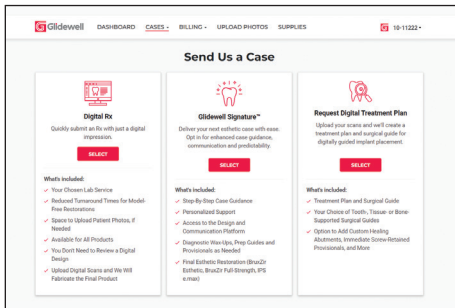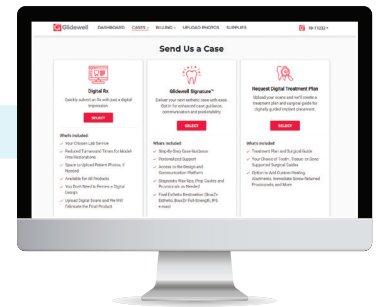

- Upload the design file and submit case via My Account (myaccount.glidewell.com)
- Select Digital Rx
- Enter Patient Information
- Enter Product Information
 - Restoration: Select Full-Arch Implant Prosthesis
 - Material: Choose between BruxZir Full-Strength Mill Service or BruxZir Esthetic Mill Service
 - Gingival Shade: Select desired gingival shade
 - Tooth Shade: Select desired tooth shade
- Upload final STL file

5 DAYS IN LAB TO FABRICATE BRUXZIR FULL-ARCH IMPLANT PROSTHESIS

■ DELIVERY APPOINTMENT

Delivery of the Final Prosthesis



- If used, retighten multi-unit abutments to manufacturer-recommended torque.



- Seat the BruxZir prosthesis on the implants or multi-unit abutments.



- Hand-tighten the enclosed final prosthetic screws, alternating from one side to the other.



- Tighten the screws to 15 Ncm. Wait approximately 5 minutes and retorque screws.



- Place Teflon[®] tape in each screw access channel.



- Fill screw access channels with light cure composite to prevent bacteria build-up.



- Confirm occlusion.



- Make adjustments if necessary.

A BruxZir™ Adjustment & Polishing Kit can be purchased via Glidewell Direct.



- If adjustments were necessary, polish the affected areas. Reinforce oral hygiene instructions.

■ POST-DELIVERY CHECK

- Verify occlusion
- Review oral hygiene instructions
- Establish recall schedule

■ MAINTENANCE APPOINTMENTS

How to Maintain the Final Prosthesis

Note: Removal of the prosthesis is NOT recommended. If the prosthesis is removed, prosthetic screws MUST be replaced.

- Six-month hygiene appointment
 - Perform prophylaxis under the prosthesis
- Twelve-month (annual) hygiene appointment



IMPLANT SOLUTIONS

866-861-7955 ■ glidewell.com/bip

GL-4923703-020126