
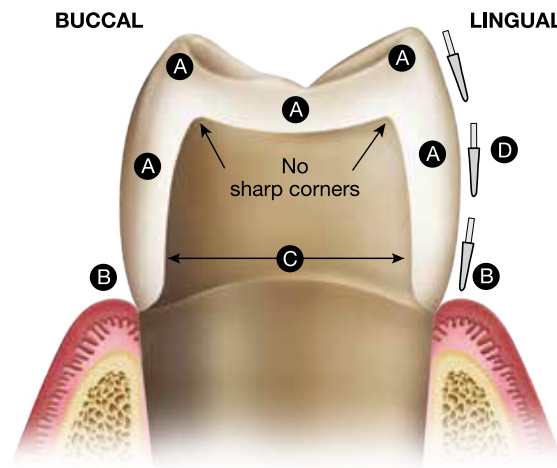


# BruxZir® *Fusion*

## PREPARATION GUIDELINES

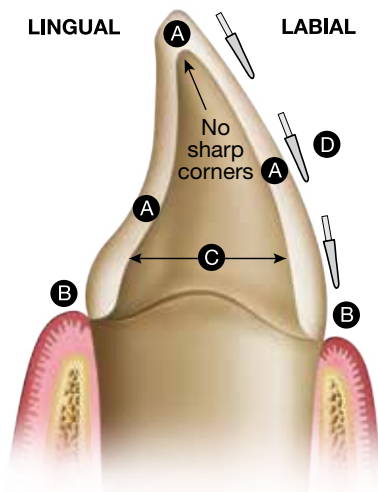

- A. 2.0 mm incisal reduction (1.75 mm minimum),  
1 mm facial reduction (0.8 mm minimum)
- B. Chamfer or shoulder margins required
- C. Rounded external walls, and axial walls must  
be convergent (avoid undercuts)
- D. 3 planes of reduction required on facial  
surfaces
- E. To achieve optimal impression quality,  
gingival retraction is necessary for  
preparations with subgingival or equigingival  
margins