


# **PATIENT HOME CARE INSTRUCTIONS**

## FOR USE WITH YOUR PARTIAL DENTURE

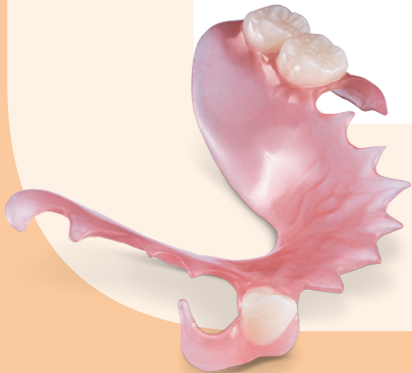



**Remove partial every night** to give your gums the chance to rest.



**Brush Daily.** Remove partial. Rinse to remove loose food or debris, then brush outside of the mouth with cold tap water and an extra-soft toothbrush.



**Soak partial** in cold tap water at least twice per week. Add Polident® or Efferdent® cleansing tablets for cleaning (follow manufacturer instructions).



**Do NOT use** toothpaste, hydrogen peroxide, baking powder, mouthwash (or other forms of alcohol), bleach, boiling water or sodium hydroxide to clean or remove stains.



**Warm-water reset:** You may soak your partial in warm water to help it regain its shape if it feels slightly distorted. Use only warm—not hot—water.



**Maintain health of remaining teeth.** See your dentist annually. If your partial breaks, chips, cracks, stains or becomes loose, contact your dentist immediately.

## ADDITIONAL TIPS

- **Do NOT modify the partial denture yourself.** Doing so can damage it beyond repair, causing fit and function issues. Self-service of your partial is evident to your dentist and will instantly void any expressed warranty.
- **Brush any remaining natural teeth twice a day** and before reinserting your partial denture.
- **If you have further questions about your partial,** please call the office to set up an appointment.

*Polident® is a registered trademark of the Hialeah group of companies. Effident® is a registered trademark of Medtech Products Inc. Instructions based on ADA recommendations.*