

Glidewell HT™ IMPLANT SYSTEM

Drilling Protocol (Ø4.3 x 13 mm)

RECOMMENDED DRILLING SPEEDS

Twist/Shaping Drills: 800-1200 RPM

Screw Tap: Do not exceed 25 RPM



Consult Instructions for Use

Twist Drill
Ø1.5 x 8 mm



Direction
Indicator
Ø3.5 mm



Twist Drill
Ø2.4/1.5 mm



Twist Drill
Ø2.8/2.4 x 13 mm



Shaping Drill
Ø3.5 x 13 mm



Final
Shaping Drill
Ø4.3 x 13 mm




Direction
Indicator
Ø4.3 mm



Optional-Dense Bone
Screw Tap
Ø4.3 mm



Implant Driver
Ø3.5/4.3
Short/Long

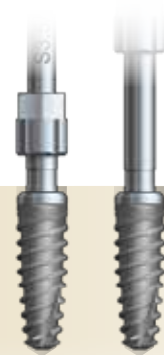

Glidewell HT™
Implant
Ø4.3 x 11.5 mm





Ensure all surgical instruments are available prior to surgery. Osteotomy depth should be increased incrementally to ensure proper guidance of the surgical tools. Do not use any drill that exceeds the diameter or length of the prescribed implant.

Glidewell HT™ Drilling Protocol (Ø4.3 x 13 mm)
IMPLANT SYSTEM

Rx Only By Prescription Only

	DESCRIPTION	CODE
	Twist Drill Ø1.5 x 8 mm	70-1071-SRG0266
	Direction Indicator Ø3.5 mm	70-1071-SRG0309
	Twist Drill Ø2.4/1.5 mm	70-1071-SRG0264
	Twist Drill Ø2.8/2.4 x 13 mm	70-1071-SRG0271
	Shaping Drill Ø3.5 x 13 mm	70-1071-SRG0276
	Shaping Drill Ø4.3 x 13 mm	70-1071-SRG0281
	Direction Indicator Ø4.3 mm	70-1071-SRG0310
	Screw Tap Ø3.5 mm	70-1071-SRG0293
	Implant Driver Ø3.5/4.3 Short	70-1071-SRG0297
	Implant Driver Ø3.5/4.3 Long	70-1071-SRG0298
	Glidewell HT™ Implant Ø4.3 x 13 mm	70-1189-IMP0012

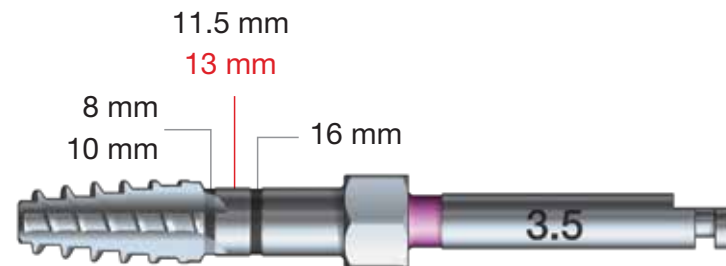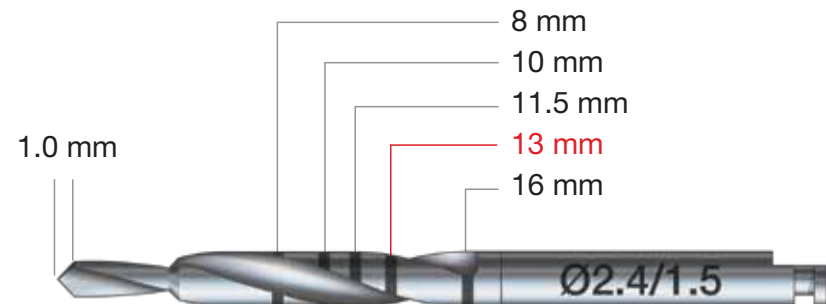

glidewell-ht.com