

BruxZir® NOW

“The Strength You Trust”



MAXIMUM
STRENGTH



NO OVEN
NEEDED



MINIMAL
PREPARATION



THE ONLY FULLY SINTERED ZIRCONIA FOR IN-OFFICE USE

- Recommended for posterior crowns demanding maximum strength
- Mills in under 40 minutes*
- Available exclusively for Glidewell milling systems
- The No. 1 brand of full-contour zirconia, with over 20 million restorations prescribed worldwide

Before



After

This patient presented with failing restorations and recurrent caries on teeth #12-15. The replacement BruxZir® NOW restorations are esthetic, comfortable and strong enough to last the patient long into the future.

“

*I'm absolutely thrilled about BruxZir NOW. As the first chairside material that is virtually immune to fracture, I've found that BruxZir NOW crowns require less handling than other in-office solutions. I simply cut off the sprue, polish and cement. **Best of all, BruxZir allows a minimal reduction prep design.**”*

– Perry Jones, DDS, MAGD | Richmond, VA

For more information
glidewelldental.com | 844-949-7184

 **Glidewell**
for the sake of smiles

BENEFITS

- Deliver restorations right from the mill with no post-processing required
- Easy delivery — conventional cementing with no need to bond
- Preserve your patients' existing dentition with minimal preparation requirements

SPECIFICATIONS

MATERIAL

- Pre-shaded, fully sintered zirconia
- Flexural strength: 1,100 MPa


IDEAL FOR

- Single-unit posterior restorations
- Cases with limited interocclusal space
- Restorations with < 1 mm margin thickness


SHADES

- 10 VITA® Classical Shades: A1, A2, A3, A3.5, B1, B2, C1, C2, D2, D3
- Posterior blocks also available in: A4, B3, B4, C3

READY TO DELIVER IN UNDER 45 MINUTES

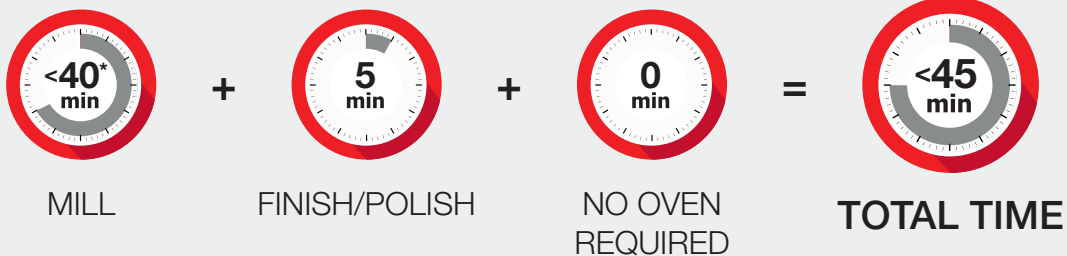

EASY PREPARATION

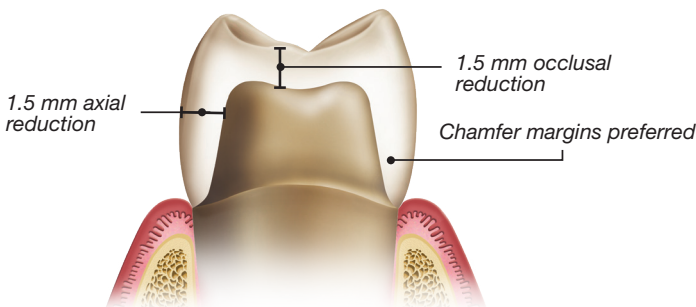

- 0.7 mm incisal/occlusal reduction required (1.5 mm recommended)
- Chamfer or shoulder margins preferred; feather-edge OK
- All axial walls must be convergent

For more information
glidewell dental.com | 844-949-7184



*Milling times vary based on prescription and mill used. Average milling time shown reflects use of the fastmill.io™ In-Office Mill. VITA is a registered trademark of VITA Zahnfabrik.