

## Clinical Workflow


### First Appointment: Preliminary Impressions

Take preliminary impressions of edentulous arch and opposing dentition, as well as the denture to be replaced.



### Second Appointment: Jaw Relation Records and Shade Selection

Obtain jaw relation records. Select denture tooth shade and mold.

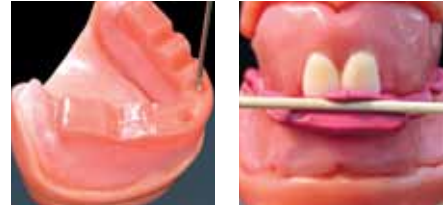

### Third Appointment: Trial Denture Setup Try-in

Try in denture. Evaluate CR, VDO, occlusion, esthetics, and phonetics. Alter setup as needed. Document with photos.



### Fourth Appointment: Verification Jig and Final Impressions

Lute implant verification jig sections. Take final impression.



### Fifth Appointment: Implant Bar Try-in

Seat and verify fit of the milled implant bar. Evaluate CR, VDO, occlusion, esthetics, and phonetics.



### Sixth Appointment: Delivery of Final Prosthesis

Deliver final prosthesis.

Follow up in one week to change out the black Locator processing caps and provide final caps with desired retention.

